

OLYMPUS


Your Vision, Our Future

Endo**Therapy**

Single-Use Electrosurgical Knife KD-625

HOOKKNIFE J

New Jet Function to Help Shorten Procedure Time

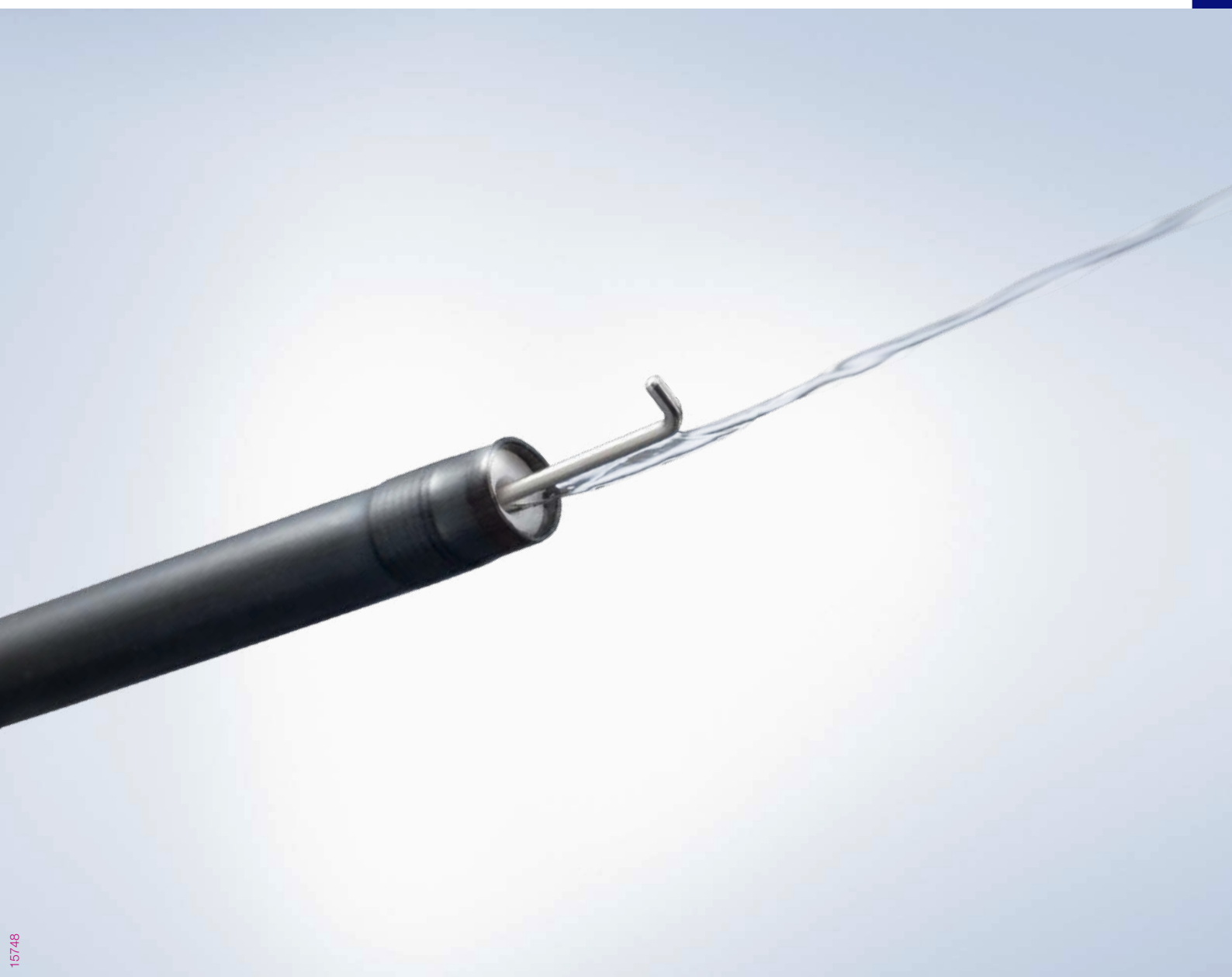

ADVANCING THE EVOLUTION OF HOOKKNIFE J

Submucosal Injection Capability Supports More Reliable and More Effective ESD Performance

Now the HookKnife does even more. With the addition of a convenient jet, the new and evolved HookKnife J will help you work and reduce the time it takes to perform an ESD procedure.



Submucosal Injection

The addition of a new jet function lets you perform saline injection after cutting – without replacing the device.

The new HookKnife J is designed to help you take advantage of the hooking technique. Using this technique will help you dissect fibrotic tissue in a narrow lumen with precision and control – even from a perpendicular direction.

Distal L-Shaped Hook

Ideal for hooking tissue and pulling it away from the mucosa and proper muscle layer, even from a perpendicular direction.

Rotation and Lock Function


Rotatable when the handle is moved slightly back. Locked when the handle is maximally extended.



Two Versions

Designed for use in the upper and lower gastrointestinal tract.


Example of Flow of ESD Procedure Using the HookKnife J

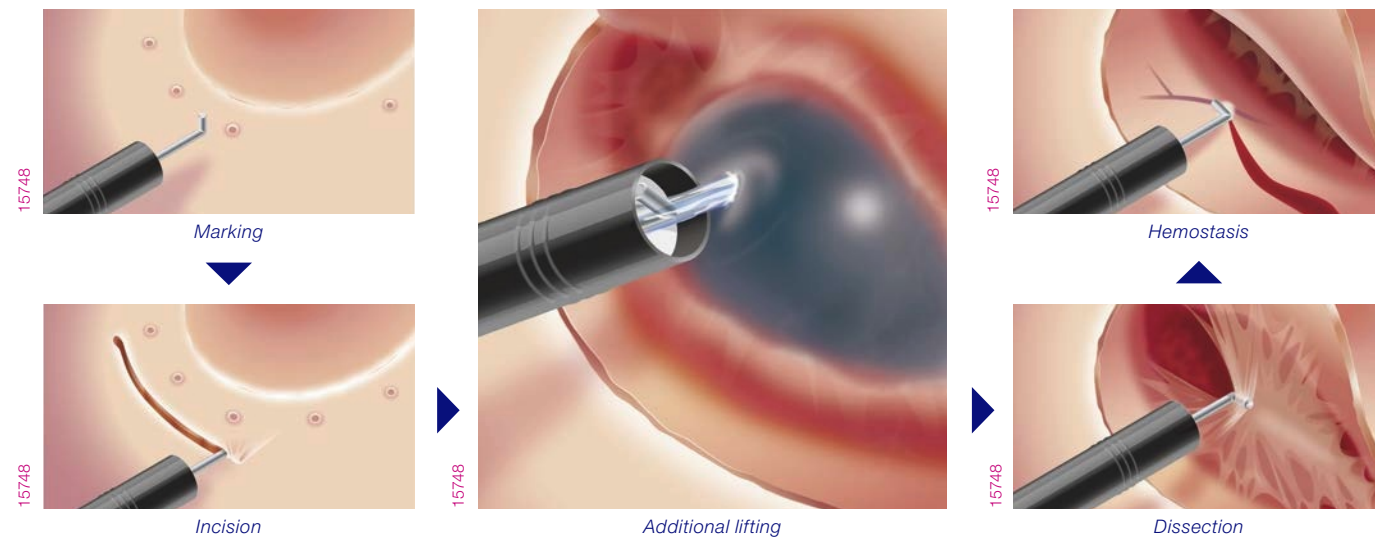

How to Use the HookKnife J – Please refer to the instructions provided with the device for instruction in details.



SINGLE-USE ELECTROSURGICAL KNIFE KD-625

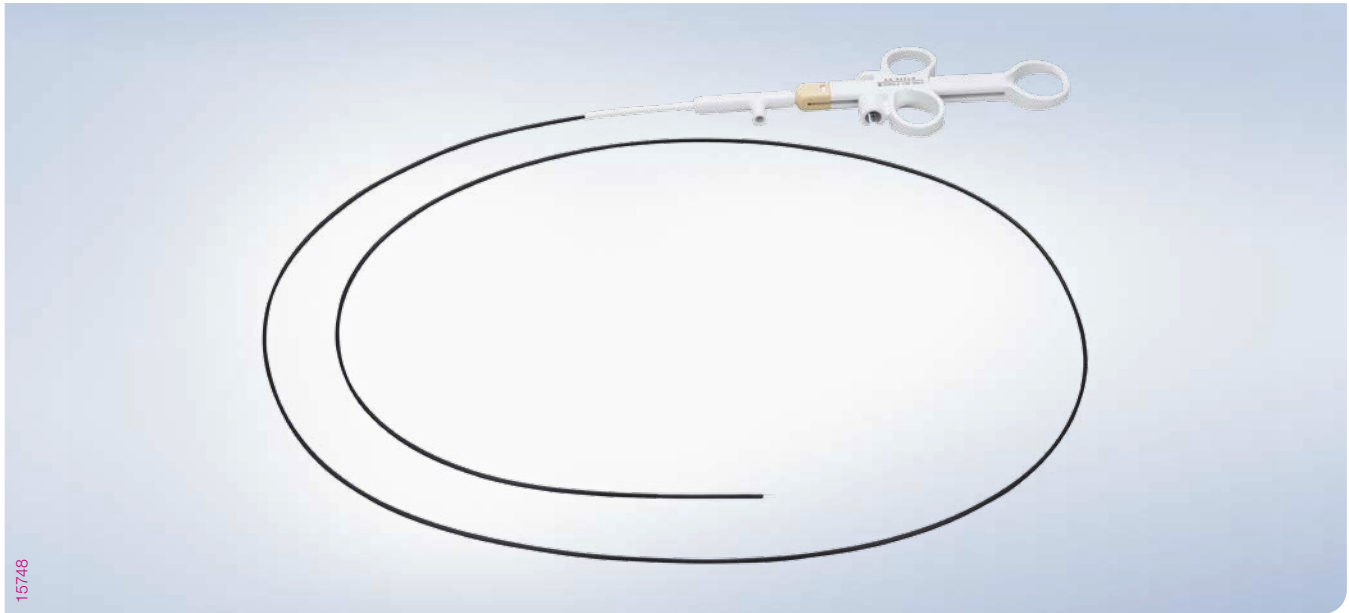

15748

Specifications

Model	Applicable channel diameter	Working Length	Cutting knife length	Sterile, single-use	Built-in handle	Length of Hook
KD-625LR	2.8 mm or more	1650 mm	4.5 mm	Yes	Yes	1.3 mm
KD-625QR	2.8 mm or more	1950 mm	4.5 mm	Yes	Yes	1.3 mm
KD-625UR	2.8 mm or more	2300 mm	4.5 mm	Yes	Yes	1.3 mm

Other Products that Will Help Make ESD Procedures Using the HookKnife J Even More Efficient



15748

Endoscopic Flushing Pump OFF-2



15748

Accessory port tube with bottle cap MAJ-1681



15748

When the MAJ-1681 is connected to the OFF-2



15748

Electrosurgical Generator ESG-100

Specifications, design, and accessories are subject to change without any notice or obligation on the part of the manufacturer.

OLYMPUS

OLYMPUS EUROPA SE & CO. KG
 Postbox 10 49 08, 20034 Hamburg, Germany
 Wendenstrasse 14-18, 20097 Hamburg, Germany
 Phone: +49 40 23773-0, Fax: +49 40 233765
 www.olympus-europa.com



E0428320EN · XXX · 01/17 · PR · ED